



**HealthCare**  
by ICP

# Fermacidal Med

Disinfectant liquid for dental and surgical instruments

CE 0068

SWISS  
MADE



## Excellent disinfection for dental & surgical instruments

Ideal also for optical and metallic components

### ADVANTAGES

- Excellent disinfection performance with a broad microbiological efficacy spectrum.
- Free from alcohol, formaldehyde, phenol, chlorine and metals.
- Gentle on the skin, does not cause irritation.
- Excellent material compatibility with stainless steel, polycarbonate (PC), silicone, polypropylene (PP), artificial leather, and more.
- MEA (Mouse Embryo Assay) and SSMA (Sperm Survival & Motility Assay) tested.
- VOCs-free.
- Non-flammable.
- Odourless.

### PRODUCT DESCRIPTION

ICP's alcohol-free dental & surgical instrument disinfectants provide reliable protection against bacteria, viruses, and yeasts. Their innovative formulation ensures they do not damage the sensitive surfaces of medical devices.

Manufactured to the strictest **Swiss quality** standards, our formulas guarantee effective disinfection. At the same time, our products protect without harming the skin, preventing dryness or irritation – an essential advantage in environments where hygiene, safety, and efficiency are all required.



## AREAS OF APPLICATION

Some examples are:

- **Medical care providers:** Hospitals, outpatient clinics, ambulatory centers, emergency rooms, mobile medical units.
- **Dental care providers:** Dental clinics, orthodontic practices, oral surgery centers.
- **Specialized & preventive care:** Care centers, medical practices, physiotherapy and massage centers, podologies.
- **Laboratories & research institutes:** IVF labs, cell labs, laboratory equipment such as: CO<sub>2</sub> incubators, laminar flow hoods, benchtop warmers, multiwell plates, pipettes and holders.
- **Pharma & medical supply chain:** Pharmacies, medicine manufacturing, medical and chemical suppliers.
- **Health Support & Wellness:** First responder services, medical training, beauty and nail salons.

## MICROBIOLOGICAL EFFICACY

| Spectrum of activity                   | Microbiology           | Contact time |
|----------------------------------------|------------------------|--------------|
| Bactericidal<br>EN 13727, EN 16615     | Staphylococcus aureus  | 1 min        |
|                                        | Enterococcus hirae     |              |
|                                        | Escherichia coli       |              |
|                                        | Proteus mirabilis      |              |
|                                        | Pseudomonas aeruginosa |              |
| Yeasticidal<br>EN 13624, EN 16615      | Candida albicans       | 1 min        |
| Limited spectrum virucidal<br>EN 14476 | Adenovirus             | 15 min       |
|                                        | Norovirus              |              |

New tests in progress

## PRODUCT ITEM LIST

| FERMACIDAL MED |                    |                 |
|----------------|--------------------|-----------------|
| Packaging      | Form               | Art. N.         |
| 250 ml         | Spray bottle       | FM_LB0250_HC_EH |
| 500 ml         | Spray bottle       | FM_LS0500_HC_EH |
| 1 l            | Spray bottle       | FM_LB1000_HC_EH |
| 5 l            | Canister           | FM_LCA005_HC_EH |
| 10 l           | Canister           | FM_LCA010_HC_EH |
| 70 pcs         | Wipes (26 x 30 cm) | FM_WBU070_HC_EH |

Ready to use

Please note that the availability of products may vary in different countries and regions. Contact your local distributor partner for more information. The recommendations regarding our preparation are based on scientific tests and are given in good faith. More detailed recommendations, e.g. regarding material compatibility or microbiological efficacy are possible, only in separate and individual cases. Our recommendations are not binding and do not constitute a guarantee. They do not preclude a company's own testing for the intended purpose and process. In this respect we cannot accept any liability. This is in accordance with our general condition of sale and supply. Image from Shutterstock.

## DIRECTION FOR USE

Use the product undiluted. Apply the ready-to-use solution by spraying it onto a wipe, or use a pre-impregnated wipe, to treat the smooth surface of the medical device previously cleaned of any visible contamination. Ensure the entire surface is completely covered and remains visibly wet for the full prescribed contact time. For electrical equipment, apply only after disconnecting from the power supply. Do not use for the final disinfection of critical or semi-critical medical devices. Avoid direct contact with food and beverages. For professional use only. Always read the label carefully before use. Report any serious incidents related to the product to the manufacturer and the competent authority.

## STORAGE

FERMACIDAL MED disinfectant must be stored in its original, intact packaging, carefully closed after use, in a dry, cool place at a temperature between 15 °C and 30 °C, and protected from direct sunlight.

## ACTIVE INGREDIENTS

Quaternary ammonium compounds.

## MEA AND SSMA TESTED

FERMACIDAL MED is MEA and SSMA tested.

## MATERIAL COMPATIBILITY

Excellent material compatibility with stainless steel, polycarbonate (PC), silicone, polypropylene (PP), artificial leather, and more.

## CLASSIFICATION

Medical device class IIa (liquid and wipes)

## WARNING AND PRECAUTIONARY STATEMENTS

H412 Harmful to aquatic life with long lasting effects

P102 Keep out of reach of children

P273 Avoid release to the environment

P501 Dispose of contents/container to in accordance with local/regional/national/international regulations

EC REP **IC PRODUCTS HUNGARY Kft,**

Zöldomb utca 36-38 C, 1025 Budapest, Hungary.

