



**Public Services**  
by ICP

# Icepur

Disinfectant cleanser concentrate liquid

SWISS  
MADE



## Excellent disinfection and cleaning for instruments and surfaces

**Concentrate**

### ADVANTAGES

- Excellent disinfection and cleaning performance, with a broad microbiological efficacy spectrum.
- Free from alcohol, formaldehyde, phenol, chlorine and metals.
- Effectively dissolves dirt, proteins, fats, and dried blood stains.
- Excellent material compatibility with stainless steel, aluminum, polycarbonate (PC), silicone, polypropylene (PP), artificial leather, and more.
- VOCs-free.
- Non-flammable.
- Odourless (optional with scent).

### PRODUCT DESCRIPTION

**ICP's alcohol-free instrument and surface disinfectant** cleansers provide reliable protection against bacteria, viruses, and fungi. Their innovative formulation ensures they do not damage sensitive surfaces, furniture, flooring, instruments, or work equipment.

Manufactured to the strictest **Swiss quality** standards, our formulas guarantee effective disinfection. At the same time, our products protect without harming the skin, preventing dryness or irritation – an essential advantage in environments where hygiene, safety, and efficiency are all required.



## AREAS OF APPLICATION

Some examples are:

- **Public administration & safety:** Governmental institutes, police, military, civil protection.
- **Education facilities:** Nurseries, kindergartens, schools, higher education institutions, research institutes.
- **Infrastructure & utilities:** Railways, aviation, maritime, technical infrastructure services.
- **Hospitality & wellness:** Hotels, restaurants, fitness, wellness centers, pools, malls.
- **Facility & services management:** Public facilities, facility management companies.

## MICROBIOLOGICAL EFFICACY

| Spectrum of activity                          | Microbiology             | Contact time |
|-----------------------------------------------|--------------------------|--------------|
| <b>Bactericidal</b><br>EN 1276, EN 13697      | Staphylococcus aureus    | 15 min       |
|                                               | Enterococcus hirae       |              |
|                                               | Escherichia coli         |              |
|                                               | Pseudomonas aeruginosa   |              |
| <b>Fungicidal</b><br>EN 1650, EN 13697        | Candida albicans         | 15 min       |
|                                               | Aspergillus brasiliensis |              |
| <b>Limited spectrum virucidal</b><br>EN 14476 | Norovirus                | 30 min       |
|                                               | Adenovirus               |              |

New tests in progress

## PRODUCT ITEM LIST

| ICEPUR    |          |                    |
|-----------|----------|--------------------|
| Packaging | Form     | Art. N.            |
| 2 l       | Bottle   | IC_LB2000_PS_EH    |
| 2 l       | Bottle   | IC_LB2000_PS_EH/LM |
| 5 l       | Canister | IC_LCA005_PS_EH    |
| 5 l       | Canister | IC_LCA005_PS_EH/LM |
| 10 l      | Canister | IC_LCA010_PS_EH    |
| 10 l      | Canister | IC_LCA010_PS_EH/LM |

Concentrate

LM: with lemon scent

## MATERIAL COMPATIBILITY

Excellent material compatibility with stainless steel, aluminum, polycarbonate (PC), silicone, polypropylene (PP) and artificial leather, and more.

Please note that the availability of products may vary in different countries and regions. Contact your local distributor partner for more information. The recommendations regarding our preparation are based on scientific tests and are given in good faith. More detailed recommendations, e.g. regarding material compatibility or microbiological efficacy are possible, only in separate and individual cases. Our recommendations are not binding and do not constitute a guarantee. They do not preclude a company's own testing for the intended purpose and process. In this respect we cannot accept any liability. This is in accordance with our general condition of sale and supply. Image from Shutterstock.

## DIRECTION FOR USE

Concentrate to be diluted with water as directed. For a 2% solution, mix 20 ml of ICEPUR with 1 liter of water. Apply the diluted product evenly to the non-porous surface or equipment to be cleaned and disinfected, ensuring complete coverage. Keep the surface moist for the entire specified contact time. For electrical equipment, apply only after disconnecting from the power supply. After the contact time has elapsed, rinse thoroughly. Avoid direct contact with food and beverages. Suitable for professional users and the general public. For safe use, always read the label before use. Report any serious incidents related to the product to the manufacturer and the competent authority.

## STORAGE

ICEPUR disinfectant cleanser must be stored in its original, intact packaging, carefully closed after use, in a dry, cool place at a temperature between 15 °C and 30 °C, and protected from direct sunlight.

## BIOCIDAL ACTIVE INGREDIENTS

Quaternary ammonium compounds.

## REGISTRATION

- BAG N. CHZB0208 (liquid)
- BAuA N-24414 (liquid)

## PRODUCT TYPES

- PT2: Disinfectants and algacides not intended for direct application to humans or animals
- PT3: Veterinary hygiene
- PT4: Food and feed area

## CLASSIFICATION

Concentrate: GHS09 Signal word: Attention.

## WARNING AND PRECAUTIONARY STATEMENTS

H314 Causes severe skin burns and eye damage

H411 Toxic to aquatic life with long lasting effects

P102 Keep out of reach of children.

P280 Wear protective gloves/eye protection

P301+P330+P331 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing

P501 Dispose of contents/container in accordance with local regulations

